

RoboComb⁺

automated operation device



60RBC200



Biogal Galed Labs.

YOUR
DIAGNOSTIC
COMPANION



Simple walk away operation In veterinary practices

- ▶ LCD touch screen
- ▶ Multi Languages
- ▶ Built in Plate Warmer
- ▶ Trusted Biogal Accuracy
- ▶ Protective Cover
- ▶ Magnetic Comb and paper strip holder
- ▶ Digital Instructions For Use
- ▶ Factory Calibrated



RoboComb⁺
automated operation device




Scan QR



Working with RoboComb+ is simple

1 Select the desired test

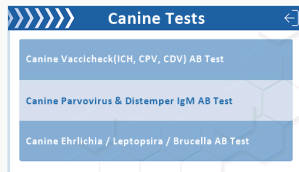

2 Bring the plate to room temperature by:

1. Leaving out of refrigeration for 60 minutes or
2. Using the built-in plate warmer for 10 minutes

3 Load the sample into the kit's plate and place the plate on the plate base of the RoboComb+


4 Load the tooth, teeth or full comb on the comb holder

1-12 teeth can be tested simultaneously



5 Let the test run

6 Read the results

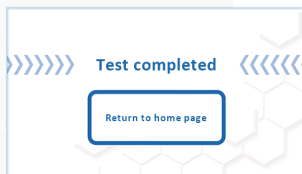



Designed for Biogal's antibody tests

COMPATIBLE WITH BIOGAL'S TESTS:

VACCICheck
by Biogal Galed Labs

Validate your Patient's protection

Scan QR



- Infectious Hepatitis
- Parvovirus
- Distemper



- Panleukopenia
- Herpesvirus
- Calicivirus

ImmunoComb
by Biogal Galed Labs

Antibody levels assessment

Scan QR



- Parvovirus & Distemper IgM
- Ehrlichia
- Leptospira
- Brucella



- Toxoplasma & Chlamydophila
- Coronavirus (FCoV)



- Brucella
- Leptospira
- Neospora



- IBD-ND-IB
- Mycoplasma Mg-Ms-Mm



Automate your test reading

CombCam

Objectively reads VacciCheck / ImmunoComb Antibody test results



Scan QR



Biogal Galed Labs.

YOUR
DIAGNOSTIC
COMPANION



www.biogal.com / info@biogal.com

Distributed by Abacus dx

1800 ABACUS (AUS) 0800 222 170 (NZ) | info@abacusdx.com | www.abacusdx.com

abacus dx