

IDYLLA™ THYROIDPRINT® ASSAY

First-in-class cartridge-based
assay for risk stratification of
indeterminate thyroid nodules



GENEPRO DX

Distributed by
BIO CARTIS

**THINK IDYLLA™
BECAUSE TIME MATTERS**

Idylla™ ThyroidPrint® Assay is for Research Use Only, not for use in diagnostic procedures.

INDETERMINATE THYROID NODULES SCIENTIFIC BACKGROUND

Thyroid nodules are a **frequent** condition, affecting up to 30-40% of the adult population. Although most thyroid nodules have little clinical significance, in many cases a fine needle aspirate (FNA) biopsy will be performed to determine its nature. In 70% of cases, an FNA* will be reported as benign and in 10% of cases as cancer, based on cytological examination.

However, in the remaining **20%** of cases, the thyroid nodule will be reported as **indeterminate**. Thyroid cytology is reported based on the 2023 Bethesda System for Reporting Thyroid Cytopathology¹. The cytology report is considered indeterminate if the report is called as atypia of undetermined significance (AUS) (**Bethesda III**) or follicular neoplasm (FN) (**Bethesda IV**)¹.

INTRODUCING IDYLLA™ THYROIDPRINT® ASSAY

Idylla™ ThyroidPrint®, developed by GeneproDX, assesses the gene expression profile of FNA* samples to enable laboratories to conduct further research on risk stratification of indeterminate thyroid nodules.

PRELIMINARY PERFORMANCE DATA

LDT vs Idylla™	ThyroidPrint® LDT**2	Idylla™ ThyroidPrint® Assay ³
Sensitivity	91%	92%
Specificity	88%	82%
NPV	95%	96%
PPV	78%	66%

The high sensitivity and specificity in combination with the ease of use and short turnaround time make the Idylla™ ThyroidPrint® Assay first-in-class for risk stratification of indeterminate thyroid nodules.

*FNA: Fine Needle Aspirate

**Laboratory Developed Test

The Idylla™ ThyroidPrint® Assay is for Research Use Only, not for use in diagnostic procedures.

IDYLLA™ THYROIDPRINT® ASSAY



Fully automated molecular testing suitable for any lab setting



Gene expression signature based on 10 target genes in combination with a proprietary algorithm



Less than 5 minutes hands-on time (HOT)
Assay turnaround time (TAT) of approx. 160 minutes



FNA sample from a thyroid nodule which has been classified as indeterminate (Bethesda III/IV)*



Catalog number:
TP0011/6

SPECIMEN REQUIREMENTS


FNA from an indeterminate thyroid nodule collected in ThyroidPrint® Collection Buffer



*Bethesda III and IV (International)/Thy3a and Thy3f (UK)/TIR3A and TIR 3B (Italian)
The Idylla™ ThyroidPrint® Assay is for Research Use Only, not for use in diagnostic procedures.

A NEW GENE SIGNATURE

The Idylla™ ThyroidPrint® Assay is a novel RT-qPCR-based molecular signature that assesses a gene expression profile from a fresh FNA sample collected from an indeterminate thyroid nodule. The signature qPCR data is analyzed by a proprietary algorithm, which reports a classifier result based on expression levels of a combination of 10 epithelial and stromal cell target genes relative to two reference genes^{2,4,5}.

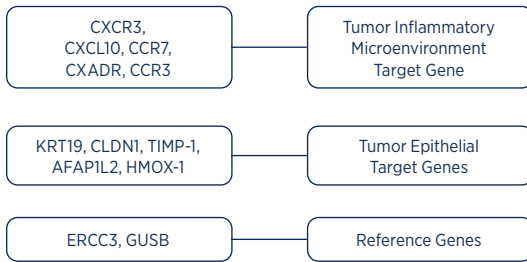

FULLY AUTOMATED SAMPLE-TO-RESULT REPORTING ON IDYLLA™ PLATFORM

Idylla™ ThyroidPrint® Assay covers the entire process from sample to result, including fully integrated sample preparation, nucleic acid extraction, reverse transcription of RNA, real-time PCR amplification and detection, data analysis, and result reporting as applicable. An automated console report will be generated with either a 'HIGH' or 'LOW' result.

A '**HIGH**' result is indicative of an **atypical gene expression**. While a '**LOW**' result is indicative of a **normal gene expression**.

A RAPID TEST - PERFORMED WITHIN 160 MINUTES IN YOUR LABORATORY

IDYLLA™ THYROIDPRINT® ASSAY



IDYLLA™ WORKFLOW



Scan sample & Cartridge




Insert sample in the Cartridge



Insert Cartridge in the Idylla™ Instrument and obtain the result within 3 hours

FIRST-IN-CLASS CARTRIDGE-BASED ASSAY FOR INDETERMINATE THYROID NODULES

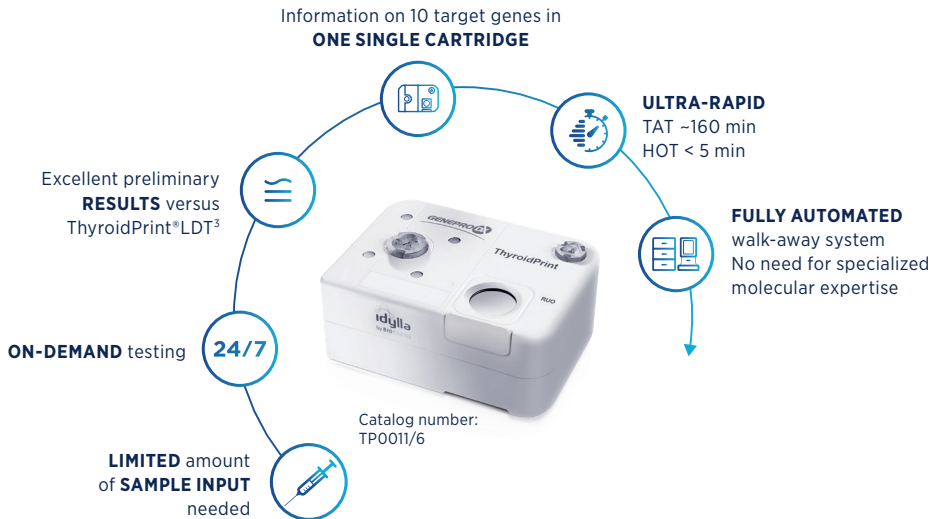

REFERENCES

(1) Ali SZ et al. Thyroid 2023; (2) Zafereo et al., Thyroid, 2020; (3) unpublished data (internal analysis based on 132 indeterminate samples compared to the surgical gold standard); (4) Gonzalez et al. Thyroid 2017; (5) Olmos et al. Endocrine-Related Cancer 2023

Biocartis NV
Generaal De Wittelaan 11B
2800 Mechelen - Belgium
+32 15 632 888

Follow us on



www.biocartis.com
customerservice@biocartis.com



Distributed by
BIOCARTIS

This product contains SuperScript™ III Reverse Transcriptase and is provided subject to a license under patents or patent applications owned by or licensed to Life Technologies Corporation, which license is limited to the human diagnostic field and research field and specifically excludes applications in forensics (including human identity testing). The SuperScript™ III trademark is owned by Life Technologies Corporation. Idylla™ platform is a CE-marked product in compliance with the EU IVD regulation 2017/746. Idylla™ ThyroidPrint® Assay is for Research Use Only (RUO), not for use in diagnostic procedures. The Idylla™ ThyroidPrint® Assay is developed by GeneProDX and distributed by Biocartis outside US. Idylla™ is available for sale in Europe, the US and many other countries. Please check availability with a Biocartis representative. Biocartis and Idylla™ are registered trademarks in Europe, the US and many other countries. The Biocartis and Idylla™ trademarks and logos are used trademarks owned by Biocartis NV. © April 2024, Biocartis NV. All rights reserved.

01USFL062ENLR204/2024

Distributed by Abacus dx

1800 ABACUS (AUS) 0800 222 170 (NZ) | info@abacusdx.com | www.abacusdx.com

abacus dx