

EASYPOINT®

retractable needle

- AUTOMATED NEEDLE RETRACTION
- EFFECTIVELY REDUCES THE RISK OF NEEDLESTICK INJURIES
- EASY TO USE - REQUIRES MINIMAL TRAINING
- ACTIVATION CAN BE ACHIEVED IN A VARIETY OF WAYS
- MAY BE ACTIVATED BEFORE OR AFTER REMOVAL FROM PATIENT
- PRE-REMOVAL ACTIVATION REDUCES THE RISK OF EXPOSURE TO THE CONTAMINATED SHARP
- ACTIVATION OF SAFETY MECHANISM AUTOMATICALLY COVERS SYRINGE LUER TIP, REDUCING RISK OF EXPOSURE TO MEDICATION, BLOOD/BODY FLUIDS
- REMAINS SAFE THROUGH DISPOSAL

compatible with both luer lock & luer slip syringes



may be used with pre-filled syringes



EASYPOINT®

RETRACTABLE NEEDLES

(50 per box/400 per case)

Now available in
NEW sizes

Catalog # Needle Size

82211	30G x ½"
82091	25G x 5/8"
82011	25G x 1"
85011	25G x 1 ½"
82031	23G x 1"
85021	22G x 1"
85031	22G x 1 ½"
85041	21G x 1"
85051	21G x 1 ½"
85061	20G x 1"
85071	20G x 1 ½"
85081	18G x 1"
85091	18G x 1 ½"

OTHER AVAILABLE SAFETY PRODUCTS

VanishPoint® Syringe
 VanishPoint® Blood Collection Tube Holder
 VanishPoint® Blood Collection Set
 VanishPoint® IV Catheter
 Patient Safe® Syringe



abacus dx



Distributed by Abacus dx
 1800 222 287 (AUS) 0800 222 170 (NZ)

EASYPOINT®

retractable needle



— EASY TO USE —

- minimal change in injection technique
- allows for single-handed activation
- **clinician's hands and fingers remain behind the exposed sharp** at all times




Attach EasyPoint needle to any luer lock or luer slip syringe. Administer medication according to institutional policy.



Apply pressure to the color-coded tab and the safety chamber using a squeezing motion, allowing the needle to automatically retract into the safety chamber. **This device is designed for activation before or after removing the needle from the patient.**



Dispose in an appropriate sharps container per protocol of institution.

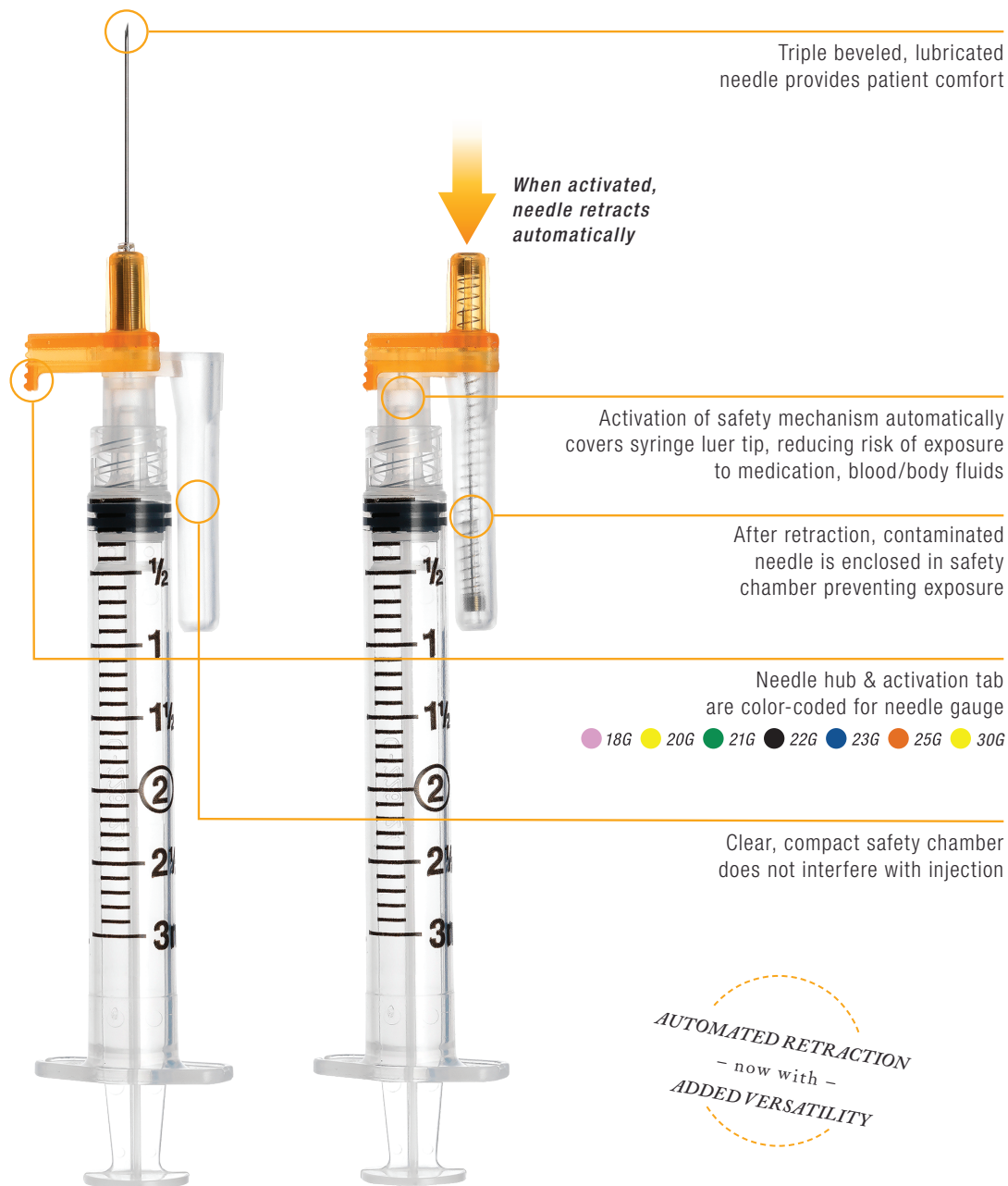

EASYPPOINT®