


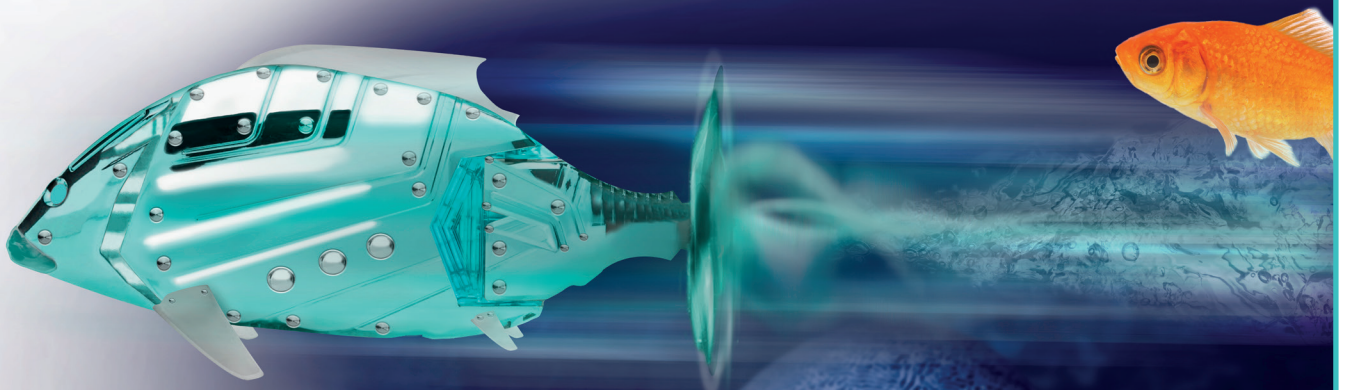

ZytoMation® *Products for automated FISH*

Fully automated Probes for the BOND™ Systems

Renowned Quality • Ready to Use • Reduced Hands-on Time



ZYTOVISION
Molecular diagnostics simplified

ZytoMation® Product Portfolio


Prod. No.	Product	Label	Tests* (Volume)
Z-2315-5.1ML	ZytoMation ALK Dual Color Break Apart FISH Probe CE [IVD]	●/●	up to 20 (5.1 ml)
Z-2306-5.1ML	ZytoMation BCL2 Dual Color Break Apart FISH Probe CE [IVD]	●/●	up to 20 (5.1 ml)
Z-2313-5.1ML	ZytoMation BCL6 Dual Color Break Apart FISH Probe CE [IVD]	●/●	up to 20 (5.1 ml)
Z-2292-5.1ML	ZytoMation ERBB2/CEN 17 Dual Color FISH Probe CE [IVD]	●/●	up to 20 (5.1 ml)
Z-2317-5.1ML	ZytoMation IGH Dual Color Break Apart FISH Probe CE [IVD]	●/●	up to 20 (5.1 ml)
Z-2321-5.1ML	ZytoMation MET/CEN 7 Dual Color FISH Probe CE [IVD]	●/●	up to 20 (5.1 ml)
Z-2312-5.1ML	ZytoMation MYC Dual Color Break Apart FISH Probe CE [IVD]	●/●	up to 20 (5.1 ml)
Z-2316-5.1ML	ZytoMation RET Dual Color Break Apart FISH Probe CE [IVD]	●/●	up to 20 (5.1 ml)
Z-2298-5.1ML	ZytoMation ROS1 Dual Color Break Apart FISH Probe CE [IVD]	●/●	up to 20 (5.1 ml)

* Using 240 µl probe solution per test. CE [IVD] only available in certain countries. All other countries research use only! Please contact your local dealer for more information.

ZytoMation® Product Information

The **ZytoMation® Probes** are compatibly designed for carrying out a fully automated Fluorescence *in situ* Hybridization (FISH) on formalin-fixed paraffin-embedded (FFPE) tissue samples ready to use on the **BOND™ systems**. To successfully use the **ZytoMation® Probes**, the **BOND™ FISH Kit (DS9636)** is required. Before starting the **BOND™ system**, the **BOND™ FISH Kit** has to be complemented by the Proteinase K solution and the **ready to use ZytoMation® FISH Probe** transferred to a titration tube placed in a container. The probe volume of the probe is sufficient for **up to 20 tests**.

Workflow Schedule

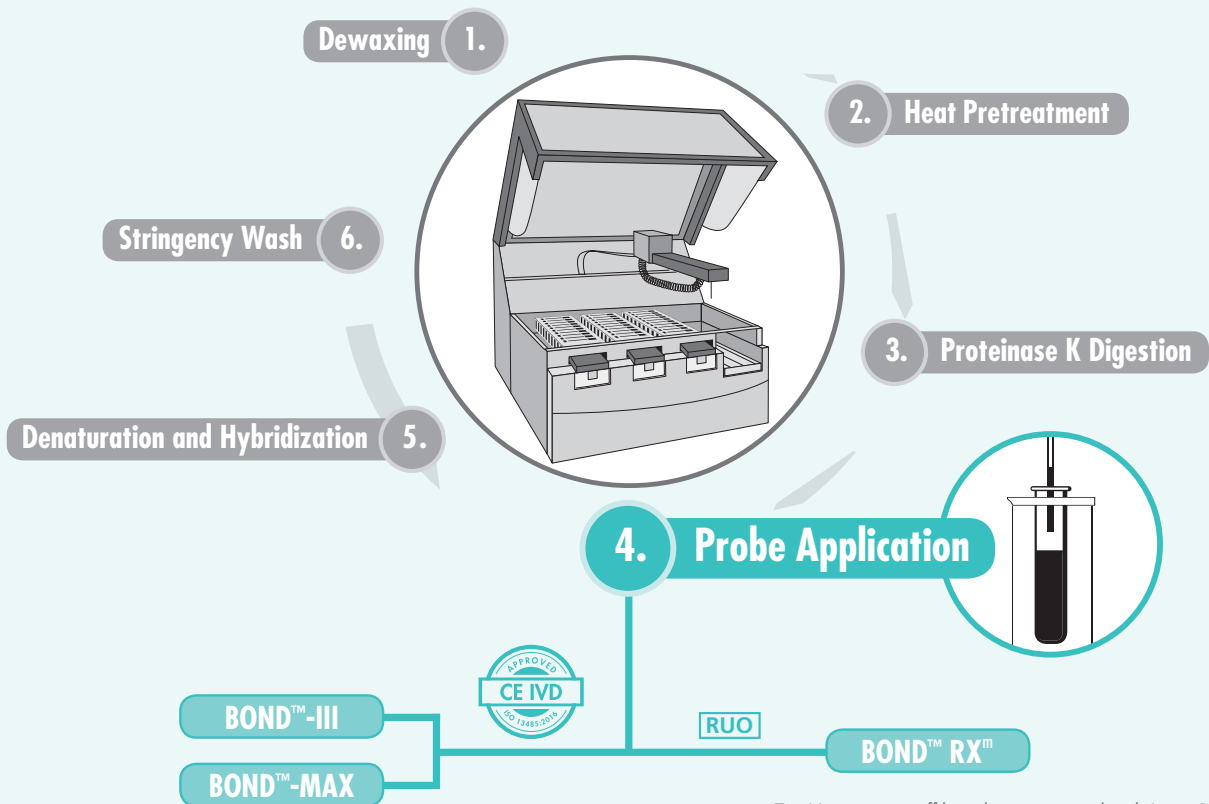

ZytoVision is not affiliated or associated with Leica Biosystems.

For more product information please contact helptech@zytovision.com or your local dealer.



ZytoVision GmbH · Fischkai 1 · 27572 Bremerhaven · Germany · www.zytovision.com

ZYTOVISION
Molecular diagnostics simplified