CutMATE

IMPROVE don't change your grossing procedures!



INNOVATIVE GROSSING FORCEPS FOR RAPID SECTIONING OF 2-3-4 MM TISSUE BLOCKS



MILESTONE

COMPASSIONATE TECHNOLOGY FOR SAME-DAY DIAGNOSIS



CutMATE Simple

The innovative forceps for that ALL important grossing and sampling of specimens



Stop hearing complaints that the tissue block was too thick and did not process!

Get CutMATE: the new forceps for simple, rapid and consistent sampling of specimens at pre-set thickness.


The CutMATE forceps were developed to overcome the variability of tissue block thickness that is heavily dependent on the type of tissue, its consistency and the grossing prowess of the operator.

The CutMate forceps, laser-shaped from special AISI 304 stainless steel, allow to cut consistently blocks of uniform thickness either 2, 3 or 4 mm.

The unique "trident-like" design of the 35 mm long prongs, ergonomically angled at 30° (for both right and left handed operator) allows cutting of tissue size to fit the standard, all plastic universal cassettes.

The inside surface of the prongs is specially textured (grooved) to provide the extra grip needed for stabilizing and slicing those diflicult, hard-to-hold tissues like breast.



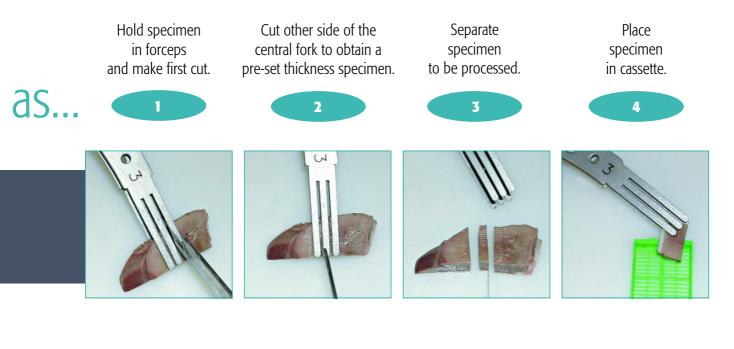


Unique features of CutMATE forceps

	Pressure points to suit specimen consistency
	Prongs are angled at 30° for an ergonomic fit
	Internal grooved contact surface texture for improved grip of specimen
0.02	Special multiple 2 mm prongs for lymph node grossing

The CutMATE advantages

- Eliminates the guess work in cutting specific block thickness
- Allows standardization of block thickness, independent of operator ability
- Improves consistency of histoprocessing due to standardization of block thickness
- Improves confidence and productivity in grossing
- Allows taking serial wet tissue blocks in one simple step (e.g. lymph nodes)
- Suitable for block's preparation for microwave and conventional histoprocessors

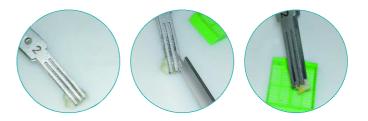

Cutting lymph nodes at 2 mm thikness

CutMATE

A range of forceps to assist the Pathologists, Pathologists Assistants, Registrars and Researchers in taking the guess-works out of getting the right size thickness block.

Accessories for user-friendly operation

Stainless steel holder for 3 CutMATE forceps

TPX container for keeping prongs submersed in sterilizing solution





CheckMATE calibration device to daily restore distance between prongs

The CutMATE forceps in operation

Ordering information

	P/N	ITEM	P/N	ITEM
0000	62362	Forcep for 2 mm tissue block	62372	TPX container (only)
0===	62361	Forcep for 3 mm tissue block	62365	Stainless steel forceps holder (only)
0:::	62360	Forcep for 4 mm tissue block	62373	Stainless steel forceps holder complete with TPX container
0000	62359	Forcep for 2, 3, 4 mm tissue blocks	62374	Calibration Device
	62357	Box with set of four (4) forceps	62375	Starter kit includes: -P/N 62357 Box with set of four (4) forceps (q.ty 1) -P/N 62373 Stainless steel forceps holder complete with TPX container (q.ty 1) -P/N 62374 Calibration device (q.ty 1)
	62358	Set of four (4) forceps (no box)		



MILESTONE S.r.l. Via Fatebenefratelli 1/5 24010 Sorisole (BG) Italy Tel. +39-035-4128264 Fax +39-035-575498 e-mail: medical@milestonesrl.com http://www.milestonemedsrl.com

MILESTONE MEDICAL 25 Controls Drive Shelton, CT 06484 - USA Tel: (203) 925-4240 Toll-free: (866) 995-5300 Fax: (203) 925-4241 e-mail: info@milestonemed.com http://www.milestonemed.com CUTIMATE/FO/REV 00/06

Distributed by Abacus ALS: AUS 1800 222 287 | NZ 0800 222 170 info@abacus-als.com | www.abacus-als.com